

cardiolife AED


Automated External Defibrillator

AED-2152K



Fighting Disease with Electronics

 **NIHON KOHDEN**

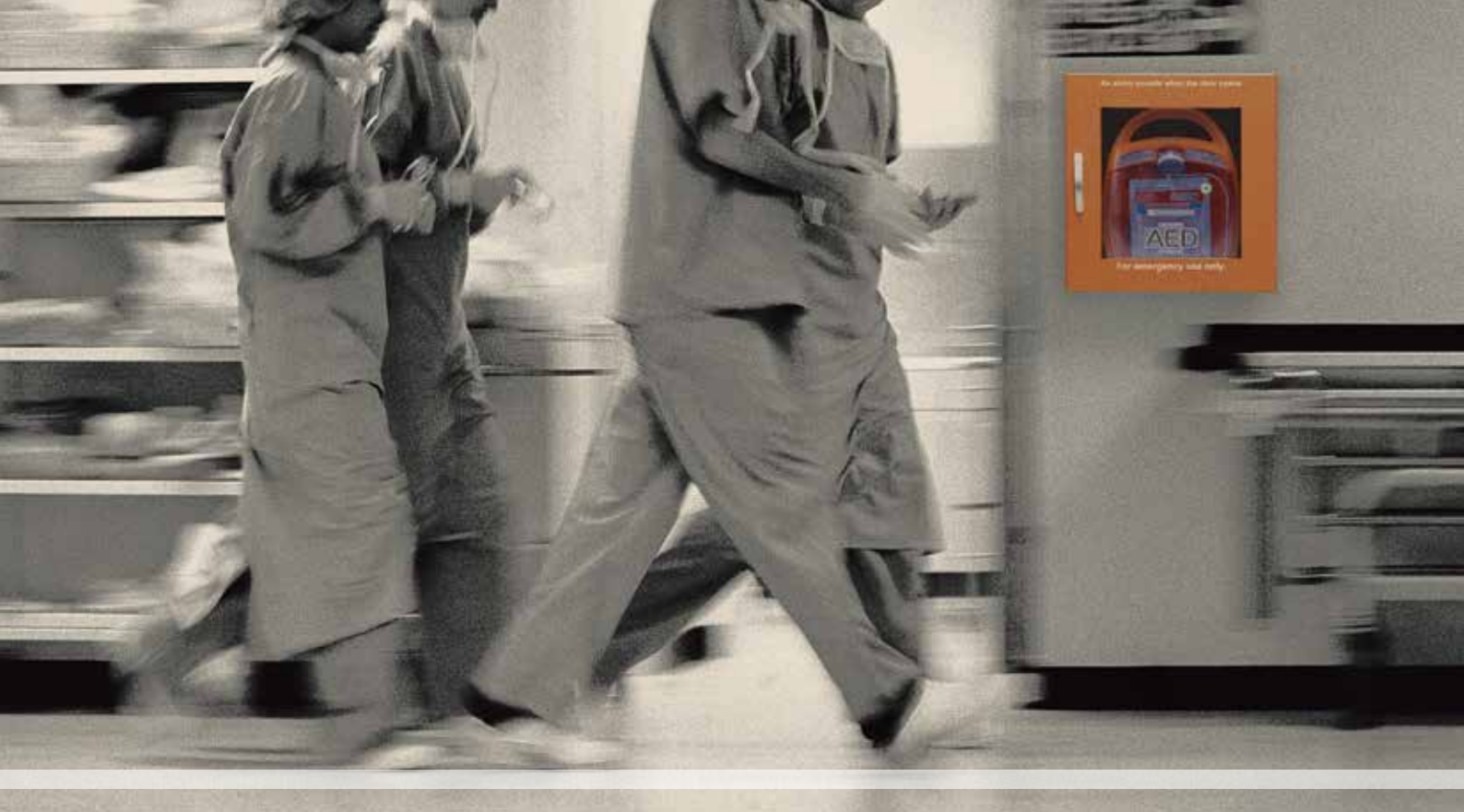

Only 3 steps to save a **life**

Step 1. Open the lid

Open the lid to turn on the power.

The voice instruction starts immediately to guide you how to operate AED.

Step 2. Attach the pads to the patient

AED pads are pre-connected, and there is no need to distinguish right from left.

Just remove the backing and press them onto the patient's chest.

Step 3. Push the shock button*

If the shock is needed, the shock button is flashing.

Push the button to deliver the electrical shock to the patient.

*In EMS mode, the operator has to decide the ECG timing analysis by pushing the analysis button.

Self test for reliability The AED has a self check function with daily and monthly tests.



System Test

Indicates current operating condition with green and red signals.



Battery Test

Indicates remaining battery life and possible trouble points.



Pads Test


Nihon Kohden pad connectors have special technology* to check and indicate the pad condition.
*patent pending






Automated External Defibrillator
AED-2152K

User-friendly onscreen operation guide

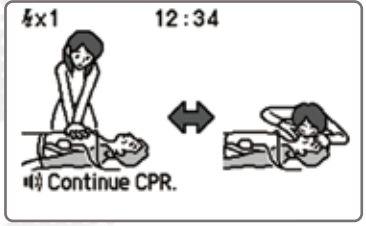

CPR instruction

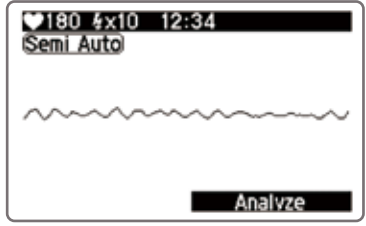

Semi auto VF analysis



AED-2152K can be used for child by only changing mode switch.

Save data and review it on a PC

AED-2152K can save 90 minutes of ECG data with event annotation. You can review the data on a PC for diagnosis.



Bluetooth interface

Rescue data and self-test results can be transferred to a PC by *Bluetooth* wireless connection.

The *Bluetooth* word mark and logo are registered trademarks owned by the Bluetooth SIG, Inc.

Consumables

- Rechargeable Battery (SB-220V)



- Battery Pack (SB-212VK)



- Battery Pack (SB-214VK)




- Defibrillation Pads (P-530)



Options

- AED Box (YZ-042H8)

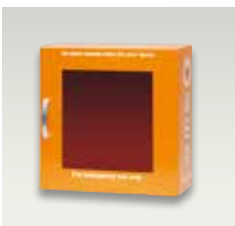

- AED Wall Mount Kit (KG-202V)



- Battery Charger (SB-205V)



- Carrying Bag (YC-210V)

- AED Report Viewer (QP-210VK)

Save a life with cardiolife AED

AED-2152K (S/N 00381 and later)
complies with RTCA/DO-160G,
Section 21, Category M.



This brochure may be revised or replaced by Nihon Kohden at any time without notice.



NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku, Tokyo 161-8560, Japan
Phone +81 (3) 5996-8036 Fax +81 (3) 5996-8100
www.nihonkohden.com