# CLEANAC•810 MK-810W

English

## General

### Intended Purpose

*For in vitro diagnostic use only.*

CLEANAC•810 is for in vitro diagnostic use as a hypochlorous acid-based detergent for the purpose of removing blood stains in order to clean the flow path of Nihon Kohden hematology analyzers.

Read the hematology analyzer operator's manual together with this manual before and during use.

NOTE: Read the SDS (Safety Data Sheet) carefully before use. The SDS is available from your Nihon Kohden representative.

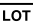
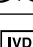
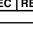
**Devices Intended for Use in Combination with**  
MEK-9100, MEK-9200

### Intended Users

For laboratory professional use only, in laboratories with suitable equipment for hematological testing. Qualified personnel, e.g. laboratory technicians trained in hematology analysis techniques, will be able to use according to this operator's manual.

## Symbols

The following symbols are used with the detergent. The descriptions of each symbol are given in the table below.

Symbol	Description	Symbol	Description
	Use by		This way up
	Lot number		Caution
	Catalogue number		Operator's manual; operating instructions
	Unique device identifier		In vitro diagnostic medical device
	Model number		Manufacturer
	Do not re-use		European representative
	Keep away from sunlight		The CE mark is a protected conformity mark of the European Union.
	Temperature limits		

## Safety Information

 **DANGER** A danger is used to alert the user to a hazardous situation which will cause death or serious injury.

 **CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in this operator's manual.

### DANGER

If the detergent contacts the eyes, wash immediately with plenty of water for at least 15 minutes and see a physician. The detergent can cause blindness.

### CAUTION

- Wear protective equipment when handling the detergent.
- Do not mix the detergent with acid. This produces chlorine gas.
- Do not inhale fumes from the detergent. If detergent fumes are inhaled, move to fresh air and take a rest.
- Do not swallow the detergent. If the detergent is swallowed or contacts the mouth, rinse the mouth immediately. Do not force vomiting. See a physician.
- If the detergent contacts the skin, wash with plenty of water. See a physician if there are skin abnormalities.

## Hazards Identification

### Signal Word

Danger

### Hazard Statement

- H290 May be corrosive to metals
- H315 Causes skin irritation
- H318 Causes serious eye damage
- H400 Very toxic to aquatic life
- H411 Toxic to aquatic life with long lasting effects
- EUH031 Contact with acids liberates toxic gas
- EUH210 Safety data sheet available on request

### Precautionary Statement Prevention

- P234 Keep only in original packaging.
- P264 Wash thoroughly after handling.
- P273 Avoid release to the environment.
- P280 Wear protective gloves, eye protection, face protection.

## Precautionary Statement Response

- P302+P352 IF ON SKIN: Wash with plenty water.
- P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P310 Immediately call a POISON CENTER/doctor.
- P321 Specific treatment.
- P332+P313 If skin irritation occurs: Get medical advice/attention.
- P362+P364 Take off contaminated clothing and wash it before reuse.
- P390 Absorb spillage to prevent material damage.
- P391 Collect spillage.

## Precautionary Statement Storage

- P406 Store in a corrosion-resistant container with a resistant inner liner.

## Precautionary Statement Disposal

- P501 Dispose of contents/container in accordance with local and national regulations.

Sodium hypochlorite: 6.3%

Sodium hydroxide: < 0.5%

## Using the Detergent

### Operation Principles

CLEANAC•810 is a detergent for hematology analyzers mainly composed of hypochlorous acid. The biggest factors that pollute the flow path of a hematology analyzer are the proteins and lipids in the blood, which fall into the "hard" category of dirt. Therefore, this product is prepared for powerful removal of such strong blood stains. It contains a hypochlorous acid that chemically dissolves blood proteins and lipids. By flowing a constant amount of this detergent through the flow path of hematology analyzer at regular intervals, dirt adhering to the flow path can be cleaned.

For details, refer to the hematology analyzer operator's manual.

### Procedure

Insert the container with the detergent into Nihon Kohden hematology analyzer. For usage instructions, refer to the hematology analyzer operator's manual.

- NOTE
- Use the detergent without diluting or sterilizing it.
  - Do not refill the detergent.
  - When using the detergent, be careful that no dust, bacteria or other contaminants enter the container.
  - The detergent is single use only.

## Technical Information

### Composition

#### Active Ingredients

Sodium hypochlorite: 6.0 to 6.3%

### Environmental Conditions

#### Storage Environment Temperature

2 to 8°C (36 to 46°F)

(Do not freeze and keep away from sunlight.)

#### Transport Environment Temperature

2 to 8°C (36 to 46°F)

The detergent can be transported at 1 to 30°C (34 to 86°F) within 4 weeks. (Do not freeze and keep away from sunlight.)

#### Operating Environment Temperature

15 to 30°C (59 to 86°F)

#### Expiration Date

The expiration date is shown on the container label and package.

## Package and Catalog Number

Model	Qty	Catalog Number
MK-810W	15 mL × 3 containers	T438R

## Disposal

When disposing of the detergent, such as when the expiration date is past, follow the instructions on the SDS of the detergent.

NOTE: Dispose of the detergent according to your local laws and your facility's guidelines for waste disposal (for incineration, melt treatment, sterilization and disinfection). Otherwise, it may affect the environment.

## Revision History

Edition	Date	Details	Code Number
1st Edition	19 Oct 2015	Initial issue	0614-907531
3rd Edition	17 Mar 2022	IVDR compliance	0614-907990
4th Edition	18 May 2022	Sodium hypochlorite concentration corrected	0604-913488
5th Edition	27 Jan 2023	Symbols updated and transport environment temperature changed	0604-913488A

- NOTE
- The code number of this manual was changed from 0614-907531A to 0614-907990 when the manual was updated from 2nd Edition to 3rd Edition and from 0614-907990 to 0604-913488 when the manual was updated from 3rd Edition to 4th Edition.
  - Changes made in the most recent edition are indicated by a bar in the left margin of each page.

Note for users in the territory of the EEA and Switzerland:  
Any serious incident that has occurred in relation to the device must be reported to the European Representative designated by the manufacturer and the Competent Authority of the Member State of the EEA and Switzerland in which the user and/or patient is established.

#### Copyright Notice

The entire contents of this manual are copyrighted by Nihon Kohden. All rights are reserved.

 Manufacturer  
**NIHON KOHDEN CORPORATION**  
1-31-4 Nishiochiai, Shinjuku-ku,  
Tokyo 161-8560, Japan  
Phone +81 3-5996-8041  
<https://www.nihonkohden.com/>

 European Representative  
**NIHON KOHDEN EUROPE GmbH**  
Raiffeisenstrasse 10, 61191 Rosbach, Germany  
Phone +49 6003-827-0 Fax +49 6003-827-599  
UK Responsible Person  
**NIHON KOHDEN UK LTD.**  
Unit 3, Heyworth Business Park,  
Old Portsmouth Road, Peasmarsh,  
Guildford, Surrey, GU3 1AF, UK  
Phone +44 14-8333-1328

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