

Battery Pack

SB-330Z

General

The SB-330Z battery pack is a backup battery for the NKV-330 ventilator.

Safety Information

- ⚠ DANGER** A danger alerts the user to a hazardous situation which causes death or serious injury.
- ⚠ WARNING** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.
- ⚠ CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ DANGER

Do not do the following to the battery pack. It may cause leakage, overheating, explosion and fire.

- Short-circuit the + and – terminals on the battery pack.
- Put the battery pack into fire or heat it.
- Disassemble or modify the battery pack.
- Give strong impact to or deform the battery pack.
- Use the battery pack on unspecified devices.
- Charge the battery pack on unspecified devices.
- Install the battery pack with the wrong polarity.
- Leave the battery pack in the reach of patients.
- Immerse the battery pack in liquid or get it wet.

⚠ DANGER

If the battery pack is damaged and the substance inside the battery pack contacts the eyes, wash immediately and thoroughly with water and see a physician. Never rub your eyes, because you may lose your eyesight.

⚠ WARNING

Remove the battery pack from the ventilator when it is not used for a long time. Otherwise the battery pack may leak.

⚠ CAUTION

Do not leave the battery pack within reach of the patient.

⚠ CAUTION

Do not use or leave the battery pack in direct sunlight or high temperatures such as in a car in hot weather or in front of a stove. It may cause leakage of the battery pack and degrade the performance and lifetime of the battery pack.

⚠ CAUTION

If the battery pack is damaged and the substance inside the battery pack contacts the skin or cloth, wash immediately with clear water. The skin may get irritated.

⚠ CAUTION

Immediately stop using the battery pack when abnormal smell, heat, deformation or discoloration is found. Otherwise overheating, smoke or explosion may occur.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
 1-31-4 Nishiochiai, Shinjuku-ku,
 Tokyo 161-8560, Japan
 Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
 Raiffeisenstrasse 10, D-61191 Rosbach, Germany
 Phone +49 6003-827-0 Fax +49 6003-827-599