

Elefix V Paste for EEG&EMG

ZV-401E, ZV-181E

General

Elefix V is a conductive paste used with surface electrodes to lower skin-electrode impedance. It can be used with electrodes for EEG and EMG examination as well as electrodes for electronic stimulation.

Safety Information

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ CAUTION

- Do not use the paste for a purpose other than stated in this manual.
- If the paste contacts the eyes or mouth, wash thoroughly and immediately with water and see a physician.
- Do not use the paste on a wound, rash, inflamed skin, infected skin, or skin eruptions, e.g. phlebitis, thrombophlebitis, varicose veins.
- A small number of patients may experience skin irritation from the use of the paste. Carefully and frequently check the skin condition of child and adult patients with sensitive skin for signs of irritation (e.g., redness, swelling) during the entire clinical procedure. If skin irritation appears, wash the paste off immediately and do not use it again.
- Do not leave the paste near the patient or in reach of children.
- If the paste is swallowed, see a physician.
- The paste is for external use only.
- Do not reuse the paste.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 11 Apr 2019
2nd Edition: 15 Apr 2020

