

# Ground Electrode

## NM-503B

### General

The NM-503B ground electrode is used in EMG/EP examination. This strap type ground electrode is covered with cloth and a hook and loop fastener. An electrode lead comes with the electrode. Use the ground electrode only with a Nihon Kohden MEB-2300A/K or MEB-9400A/K EMG/EP measuring system.

### Safety Information

**WARNING** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

**CAUTION** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

#### WARNING

When performing MRI test, remove all electrodes from the patient. Failure to follow this warning may cause skin burn on the patient. For details, refer to the MRI manual.

#### CAUTION

Connect the ground electrode only to the specified instruments. Otherwise the patient may receive electrical shock.

#### CAUTION

Do not wrap the ground electrode around a wound or inflamed skin.

#### CAUTION

Do not leave the ground electrode in water or physiological salt solution. It will rust and correct measurement cannot be performed.

#### CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer  
**NIHON KOHDEN CORPORATION**  
1-31-4 Nishiochiai, Shinjuku-ku,  
Tokyo 161-8560, Japan  
Phone +81 3-5996-8041  
<https://www.nihonkohden.com/>

 European Representative  
**NIHON KOHDEN EUROPE GmbH**  
Raiffeisenstrasse 10, 61191 Rosbach, Germany  
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 15 May 2019  
2nd Edition: 12 Dec 2022

